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Manual breast pump with air removal system

Instructions for use:

All parts of the breast pump (main body, silicon cover, rubber pump, silicon tubing and bottle including its parts) should be washed with warm soapy water and thoroughly rinsed before use. The washed parts should next be sterilised by pouring boiling water on them (the rubber pump does not usually require sterilisation). Sterilised parts have to cool down and dry out. After drying, a silicon massaging cover should be placed on the main body of the breast pump and the main body connected with the rubber pump with the help of the tubing. Next the bottle should be screwed in.

We recommend using the breast pump in a quiet and comfortable place for milk expression to proceed in a nice and peaceful atmosphere.

The main body of the breast pump should be put against the nipple in such a way that the breast nipple is placed in the centre of the main body. Holding the main body of the breast pump against the breast with one hand you should press the pump gently with the other hand in order to create suction power. The suction power of the breast pump is adjusted by the deepness and velocity of the hand pressing the pump.

If the suction power during milk expression is too strong, the air removal button should be pressed. This causes the suction to stop until the next pressing of the pump. This system imitates the way a baby sucks.

Milk expression from both breasts alternately, maximum seven minutes each, is recommended.

All parts of the breast pump should be washed again in warm soapy water, thoroughly rinsed and sterilised after each use. The sterilised breast pump should be kept in a sterile, dry, closed container.

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