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
Digital forehead and ear thermometer

**It includes: 1 forehead probe
1 stand
1 user manual**

**Please read the user manual before
using the thermometer.**

**The thermometer has a talking function
which reads out the measured temperature
(option in English)**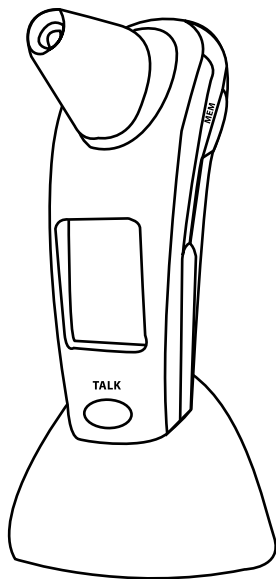

PLEASE READ THE USER MANUAL BELOW BEFORE USING THE THERMOMETER

- Thank you for purchasing our digital infrared thermometer. It is recommended to become familiar with this user manual to operate the product appropriately.
- It is recommended to read and understand the safety measures included in this user manual to operate the product safely and appropriately.
- The user manual should be kept for informational purposes.

SAFETY MEASURES

- Warning signs and symbols were used for your safety and appropriate use of the product to prevent any injuries.
- Please become familiar with the table below describing the signs and symbols:

 WARNING	This means the possibility of injury if inappropriately used.
 ATTENTION	This means the possibility of injury or damage of property if inappropriately used.

ZNAKI I LUB SIMBOL

	ATTENTION This means a warning expressed by words or a drawing – “Do not break the thermometer”.
	PROHIBITION This means a prohibition expressed by words or a drawing - prohibited.
	This means that the thermometer does not require a protective cover for the probe.
	SUGGESTION OF A SYMBOL This means B type device.

ATTENTION

Do not use the thermometer in extreme temperatures (below 15°C/59°F or above 35°C/95°F) or in extreme humidity (below 20%RH or above 80%RH).

* Not following these recommendations may cause inaccuracy of measurement.

Do not expose the thermometer to extreme temperatures (below - 10°C/14°F or above 55°C/131°F) or extreme humidity (below 20%RH or above 90%RH).

* Not following these recommendations may cause inaccuracy of measurement.

It is dangerous for patients to diagnose themselves and to treat themselves based on the result of the measurement. Make sure that you obey your doctor's requirements.

* Self-diagnosis may cause deterioration of disease.

Do not touch and do not blow the infrared sensor.

* A dirty or broken infrared sensor may cause inaccuracy of measurement.

The dirty infrared sensor should be gently cleaned with a soft and dry cloth.

* Cleaning the sensor with a tissue or a paper towel may scratch the infrared sensor and because of that cause inaccuracy of measurement.

Keep the device out of reach of children.

* Self-measurement by children may result in ear damage. If the battery or protective film is swallowed it is necessary to consult a doctor immediately.

If there is a difference in temperatures between the storage place and the measurement place, it is recommended to keep the thermometer approx. 30 minutes at room temperature (the measurement place) before taking the measurement.

* Not following these recommendations may cause inaccuracy of measurement.

ATTENTION

If ear ache occurs stop using the thermometer.

* It could damage the ear.

The product should not be used by persons suffering from ear infections, including otitis externa and tympanitis

* Affected places could deteriorate.

Do not use the thermometer in a wet ear (after swimming or bathing).

* It could damage the ear.

Do not throw batteries into fire.

* Batteries thrown into fire may explode.

RECOMMENDATIONS


- Do not use the product for any other purposes.
- The product is not water-resistant. Clean it gently with a damp cloth.
- Do not leave the product exposed to any chemical solvents, direct sun or high temperatures.
- Do not drop, trample or hit the product.
- Do not use a mobile phone near the thermometer while taking measurement.

- Follow the applicable national law in the case of used batteries and methods of waste disposal.
- If the thermometer is not used for a long time, take the batteries out.

USE

Do not use the product for any other purposes except measuring temperature in the ear or on the forehead. The thermometer can be used to measure temperature in everyone, e.g. newborn, small children and adults.

DESCRIPTION AND FUNCTIONING OF THE PRODUCT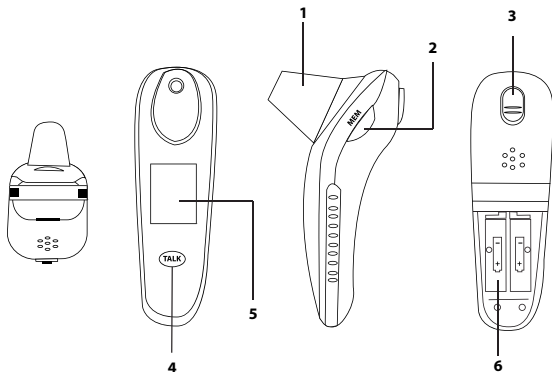


1. Forehead probe – used to measure temperature on the forehead
2. MEM key – press to read the memory of the last 30 measurements. Then press and hold for 2 s to display the setting mode.
3. Measurement key – starts measurement.
4. TALK key – to use the talking function press and hold the TALK key for approx. 2 s; the symbol of a speaker appears on the display
5. LCD display
6. Battery compartment (AAA x 2)

LCD DISPLAY

1. Day sign in time mode
2. A.M. and P.M. sign in time mode
3. Memory sign means the memory mode
4. Indicator of the series of records in memory mode
5. Year in time mode or the series of records in memory mode
6. Year sign
7. Month and day in time mode or memory mode
8. Month sign

9. Night sign in time mode
10. Sleep sign
11. Low battery
12. Temperature in measuring mode or time in time mode
13. High temperature sign
14. State of measurement sign; it flashes while measuring temperature
15. Sign of temperature measured in ear
16. Sign of temperature measured on forehead
17. Measuring temperature sign
18. SET sign means the setting mode
19. Speaker sign
20. Day sign

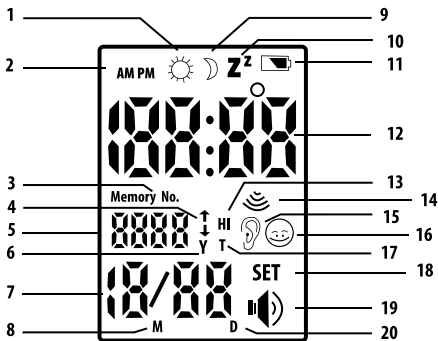
BASIC FUNCTIONS

1. Displaying current date and time: The range of displaying date according to the Gregorian calendar comprises the period of 1901~2099. The initial time is set to 1 January 2008 Rys 10
2. Measurement and temperature displaying:
3. The range of measurement: 32.0 °C (89.6 °F) – 42.9 °C (109.3 °F)
4. If the body temperature is higher than > 37.6 °C (99.6°F), the red light under the measurement key will go on to signal high temperature.
5. 30 recorded results of measurement:
6. Maximum 30 results are stored. The results of measurement are stored together with the date of measurement and sequence (ear or forehead).
7. Measurement and display of room temperature:
8. The range of measurement from 5.0°C to 59.9°C (41.0°F to 139.8°F).
9. Voice: reads the results of measurement, room temperature and current time.
10. Celsius/Fahrenheit switch: press simultaneously TALK key and measurement key to switch between °C and °F.

SETTING OF DATE AND TIME

1. Press and hold for approx. 2 s the MEM key.
2. When the hour starts flashing, each press of the MEM key permits the appropriate hour to be set.
3. Then press the TALK key to move into the minute position. With the MEM key minutes should be set appropriately.
4. The next press of the TALK key moves into the position of the date and in the same way into year, month and day.
5. At the end the flashing SLP function appears: the possibility to set the thermometer into active or sleeping mode. If we set ON after the completion of the thermometer's work with the MEM key, after approx. 3 minutes the display will turn off to save batteries. If we set OFF, the display will be active all the time.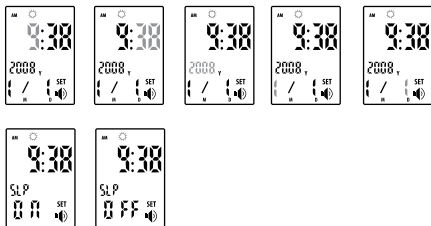

TIPS HOW TO TAKE MEASUREMENT

1. The result of measurement made in the right ear may vary from the result of measurement made in the left ear. Therefore, always measure the temperature in the same ear.
2. To achieve a precise result the ear has to be patent (this also concerns excess ear wax).
3. External factors may have an influence on the temperature in the ear, such as when the person:
 4. · lay on one or the other ear,
 5. · had covered ears,
 6. · was exposed to very high or very low temperatures,
 7. · recently swam or bathed.
8. · In such cases wait 20 minutes before measuring temperature.
9. If ear drops or other medicine were applied to the ear canal, use the ear which was not treated or measure the temperature on the forehead.

PREPARATION FOR MEASUREMENT

1. Replace low batteries to ensure maximum power.
2. Keep the thermometer for 30 minutes in the place where the measurement will take place to achieve a correct result.
3. Unexpected fluctuations of surrounding temperature may have a negative influence on the accuracy of measurement. If the place where measurement will be taken shows a different temperature than the place where the thermometer is or if an attempt is made to measure temperature near to the air conditioning, then achieving the intended result is impossible.
4. Make sure that the ear is clean so as not to cause inaccuracy in measurement.
5. The measurement should be taken while resting to ensure an accurate result. The temperature may increase directly after doing exercises or bathing.

HOW TO MEASURE TEMPERATURE

1. To measure temperature in the ear:
Take off the FOREHEAD COVER from the thermometer, place carefully the measuring probe along the ear drum into the ear, and then press the Measure key to start the measurement. You'll hear a

sound then. After the completion of measurement you'll hear two short sounds. Now you can read the result on the display or hear it through a speaker, if the TALK function is on.

2. To measure temperature on the forehead:

Put on the FOREHEAD COVER on the thermometer, place it gently in the middle of the forehead, between eyebrows and hair line. Press the Measure key to start the measurement. You'll hear a sound then. After the completion of measurement you'll hear two short sounds. Now you can read the result on the display or hear it through a speaker, if the TALK function is on.

3. The result is automatically stored in the MEMORY. If there are already 30 results there, the oldest one will be removed and the numbering will decrease by one position;

4. Measure the temperature a maximum of 3 times in a row.

If the measurement is constantly taken – 4 times or more – wait at least 10 minutes before you start another one.

5. Displaying the result of measurement:


A. Displaying the true value of the result for measurements between 32.0°C and 42.9 °C (89.6 °F – 109.3 °F);

B. "HI" displayed if the results are higher than 42.9 °C/109.3 °F;

C. "LO" displayed if the results are lower than 32.0°C/89.6 °F.

6. After the measurement is completed, after 30 seconds, the thermometer automatically returns to the time mode.

7. The display is automatically turned off 1 minute after pressing the last key (if the sleep mode is activated)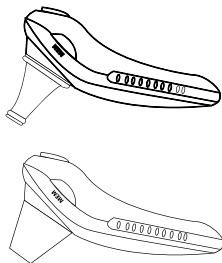


ADVANCED USE


Clock mode:

1. Time is displayed in the sequence year, month, day, hour and minute with a flashing ":" place for seconds. The LCD display shows alternately every 5 seconds the current time and room temperature.

2. Press [TALK] in the talking mode to get information on the current time and room temperature.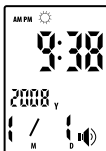

Clock mode

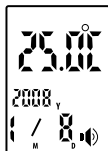
Surrounding temperature mode


Low battery mode

- Press and hold the [TALK] key for 2 seconds to choose English or non-talking option. However, during the measurement you will hear a "didi" sound, independently of whether the thermometer is in the talking mode or in the non-talking mode.
- When the TALK function is on, a speaker symbol is on. If it is turned off, the speaker symbol will not appear on the display. The default language in the TALK function is English.

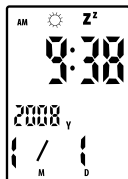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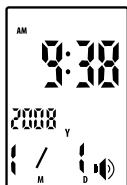Memory mode:


- Press [MEM] in the clock mode to move into the memory mode.
 - A symbol "-" will be displayed if there are no data recorded.
 - A maximum of 30 results are stored in the thermometer together with the date and the order of measurement.
 - The thermometer returns automatically to the clock mode 5 seconds after pressing the last key.
- Attention: To stop the thermometer talking, press the MEM key during the process of talking.

RECOMMENDATIONS

- It is recommended to beginner users to measure the temperature for the first time in the same ear 3 times in a row. The highest result should be recorded if there are various results because each thermometer used without a break may show a slight error in measurement (+/-0.3°C, +/-0.5°F).
- Wait approx. 30 seconds before you measure the temperature in the same person to prevent excessive cooling of the skin.
- Healthy users have almost the same temperature in both ears. Please select an ear which shows a constant and relatively higher temperature.
- Please clean the ear before measurement.

BATTERY REPLACEMENT

Remove the battery cover. Use only new alkaline batteries (2xAAA). Put the battery cover back tightly.

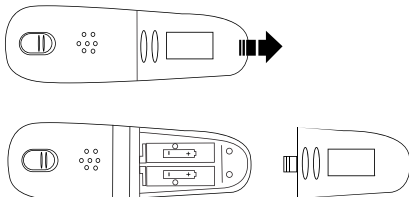MAINTENANCE, STORAGE AND CALIBRATION

Maintenance:

1. Remove all stains from the device with a soft dry cloth.
2. Clean the end of the probe as follows: wipe its surface gently with a swab of cotton wool or soft cloth moistened with alcohol. The thermometer can be used after the alcohol dries.
3. Do not wash the thermometer with water, abrasive detergent or detergent containing benzene.

Storage:

1. After each use clean the thermometer and put on the protective cover.
2. Do not leave the thermometer exposed to direct sun, high temperatures, humidity, fire, flames, shocks or hits.
3. If you do not use the thermometer for a long time, take out the batteries.

Calibration:

1. The thermometer is factory-calibrated. If the thermometer is used according to the user manual, periodic calibration is not required. If there are any doubts concerning the accuracy of measurement, please contact your shop assistant immediately. Do not try to modify or reassemble the thermometer.

PROBLEMS AND TROUBLESHOOTING:

Problems	Checklist	Action
No reaction	Dead battery? Battery in wrong polarity?	Replace the old battery Put the batteries in again, paying attention to the position of the poles
Battery symbol on the LCD display	Low battery	Replace the old battery
"Lo" on the display (result below 32°C or 89.6°F)	Was the thermometer appropriately pointed at the ear drum or forehead?	Follow the user manual which will explain how to point the thermometer at the ear drum/ forehead
"Hi" on the display (result above 42.9°C or 109.3°F).	Please check the way you use the thermometer	Follow the user manual to take a correct measurement

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TECHNICAL INFORMATION

Name of the product:	Digital infrared thermometer
Catalogue no.:	116
Power supply:	DC 3V (2 AAA batteries)
Power consumption:	< 50 mA (sound)
The part sensing temperature:	infrared sensor
Measurement range:	32.0°C to 42.9°C (89.6°F to 109.3°F)
Room temperature:	5°C to 59.9°C (41°F to 139.8°F)
Measurement accuracy:	+/- 0.2°C (or +/- 0.4°F) from 35.5°C ~ 42°C (or 95.9°F to 107.6°F) +/- 0.3°C (or +/- 0.5°F) for other ranges
Room temperature:	+/- 1°C (2°F)
Outer dimension:	120(length) x 40(width) x 65(height) mm
Weight:	90g (including batteries)
Working environment:	Temperature: 15°C to 35°C (59°F to 95°F)
Relative humidity:	20%RH ~ 80%RH
Storage environment:	Temperature: - 10 to 55°C (14°F to 131°F)
Relative humidity:	20%RH ~ 90%RH

*Do not throw used batteries away with the rubbish. Ask the shop assistant for details

LIST OF STANDARDS

This device conforms to the following standards:

ISO 10993-1:2003 Biological evaluation of medical devices -

Part 1: Evaluation and testing

ISO 10993-5:1999 Biological evaluation of medical devices --

Part 5: Tests for in vitro cytotoxicity

ISO 10993-10:2002 Biological evaluation of medical devices --

Part 10: Tests for irritation and delayed-type hypersensitivity

EN 60601-1:1990 Medical electrical equipment -

Part 1: General requirements for safety-

EN60601-1-2:2001 Medical electrical equipment -

Part 1-2: General requirements for safety – Collateral standard:

Electromagnetic compatibility – Requirements and tests

EN 12470-5:2003 Clinical thermometers-

Part 5: Performance of infrared ear thermometers (with maximum device)

WARRANTY:

The product has one-year warranty from the date of purchase. The warranty does not include damage caused by inappropriate use. This concerns:

1. Damage caused by unauthorized disassembly and modification of the device.
2. Damage caused by dropping the device while using it or transporting it.
3. Damage caused by not following the user manual.

The warranty card with the stamp of the distribution centre (including the name and address) entitles the customer to free-of-charge repair.

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Name: Shenzhen Dongdixin Technology Co., LTD.
Shanghai International Trading Corp. GmbH (Hamburg)